

Jalosome®

Soothing Gel



Class IIa medical device in gel format for the treatment of skin lesions caused by radiotherapy and chemotherapy and for the reduction of associated pain.

Intended use:

Jalosome® Soothing Gel is specifically formulated for individuals experiencing:

- Skin lesions caused by radiotherapy and chemotherapy.
- Radiodermatitis.
- Dermatitis.
- General skin lesions.
- Erythematous manifestations affecting any body area and causing pain.

Jalosome® Soothing Gel is a vesicular suspension in a medium-viscosity hydrophilic gel, **designed to treat skin lesions caused by radiotherapy and chemotherapy**. This **patented gel** consists of liposomes, vesicular microstructures that enhance the penetration of the gel's components through the stratum corneum, the skin's main barrier to permeation. This improves the gel's effectiveness, promoting skin regeneration and managing pain associated with **radiotherapy and chemotherapy**.

Instructions for Use:

One airless pump actuation, corresponding to **0.2g of gel**, is sufficient to treat an area of **16 cm² of skin**.

Apply the required amount based on the size of the lesion and gently massage to evenly spread the product. After a few minutes, no residue will remain.

Apply twice daily until the issue is resolved unless otherwise directed by a doctor.

Ingredients




Hyaluronic acid:

presente sia ad alto che a basso peso molecolare, ripara ed idrata i tessuti. Grazie alle sue proprietà gelificanti, evita l'utilizzo di altri agenti addensanti.



Allantoin:

Natural antibacterial with protective, anti-inflammatory, soothing, and moisturizing properties. It also has keratolytic action, stimulating cellular and epithelial proliferation by acting on fibroblasts and enhancing extracellular matrix synthesis.

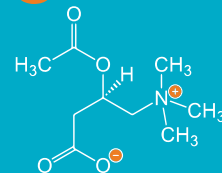

L-Acetylcarnitine:

creates conditions that promote the natural processes of cellular renewal. It works to alleviate the sensation of pain.

**SKIN LESIONS -
RADIOTHERAPY AND CHEMOTHERAPY.**

WELCARE
Protecting People - Caring for Nature

Certified
(B)
Corporation

TECHNICAL FEATURES:

- Reduces pain caused by skin lesions.
- Accelerates the re-epithelialization phase.
- Nourishes tissues.
- Non-irritating.
- Non-greasy.
- Leaves the skin soft, healthy, and smooth.
- Airless bottle, 10 ml.

INDICATION FOR USE:

- Wash hands before and after use.
- One pump delivers 0.2 g of gel, suitable for treating 16 cm² of skin.
- Apply the necessary amount based on the size of the lesion and gently massage to evenly spread the product.
- Apply twice daily until the issue is resolved, unless otherwise directed by a doctor.

CONTRAINDICATIONS:

There are no known product-specific contraindications.

SPECIAL WARNINGS AND PRECAUTIONS:

If an allergy to the components is known, avoid using the product. In case of sensitization, stop the treatment and inform your doctor. After use, do not disperse into the environment. Do not swallow.

INTERACTIONS WITH DRUGS AND OTHER FORMS OF INTERACTION:

No known interactions.

SIDE EFFECTS:

No known side effects. Do not use in case of hypersensitivity or individual allergy to the components; in case of hypersensitivity to one of the components, discontinue administration and inform your doctor.

INCOMPATIBILITY:

No known incompatibilities. For a better efficacy of the product, we do not recommend the use of other moisturizing solutions before, in conjunction with or after the use of *Jalosome® Soothing Gel*.

EXPIRY PERIOD:

The product expires in 35 months from the date of production in unopened packaging and stored correctly. Once opened, the single bottle must be used within 7 days of opening.

SPECIAL PRECAUTIONS FOR STORAGE:

Store in a cool and dry place, away from sources of heat, light and humidity.
Do not use the product if it is damaged or not perfectly intact when it is first opened.
Do not use the product after the expiry date indicated on the package.

CLASSIFICATION:

Class IIa Medical Device.



SCAN THE
QR CODE FOR
THE BIBLIOGRAPHY



PURCHASE
JALOSOME® ON
OUR WEBSITE

CODE:	DESCRIPTION	SALE UNITS:
	JALOSOME® Soothing Gel	
95810	20 airless bottles 10 ml	1 carton: 8 boxes
95810F	4 airless bottles 10 ml	1 carton: 28 boxes

Certified



B Corp Certification

It means being part of a global movement of companies that meet high standards of social and environmental impact in favor of an inclusive, equitable and regenerative socioeconomic model.
A constant commitment to the wellbeing of people, customers, communities, and the environment.

REV 00 2024/10

WELCARE INDUSTRIES S.P.A. – Benefit Corporation

Operational site: via dei Falegnami, 7 - 05018 Orvieto (TR)
Commercial office: Regus Building, piazzale Biancamano, 8 - 20121 Milano
Commercial office: EUR - Spaces, viale Luca Gaurico, 91/93 - 00143 Roma
Registered office: via San Giovanni sul Muro, 18 - 20121 Milano
www.welcareindustries.com

WELCARE
Protecting People - Caring for Nature

